

Wireless Bed and Chair Exit Notification

Unique Safety Auto-Reset Features:

TAMPER PROOF!


This Fall Alarm Monitor is the only monitor on the market with a Patent Pending feature that allows only the caregiver to silence an alarming monitor !

ALWAYS ON TECHNOLOGY!

This Fall Alarm Monitor has no "off" mode.

SIMPLE & EASY TO USE!

The Reset Button is the only button on this Fall Alarm Monitor making it easy to use. It automatically resets the Fall Alarm!

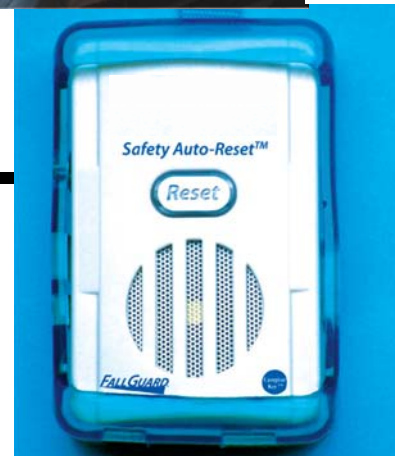

FALL ALARM SYSTEM FEATURES:

- Quick and easy installation
- Affordable Price
- Can be used with a wireless call cord
- Can be used with a *VisionLink* or *MicroVision* system

PRESSURE PADS

- Incontinence protection pads are moisture sealed
- Easy to clean

SAFETY AUTO-RESET™ FALL ALARM MONITOR

- The Caregiver Key™ (patent pending) allows only the caregiver to turn off the alarming monitor, making it **TAMPER PROOF!**
- Reset button is the only button on the monitor for easy operation
- Monitor has no "Off" mode



This Fall Alarm System consists of weight-sensitive pressure pads for bed, floor and chair use. When the pad is connected to a fall alarm monitor, the pressure pad will trigger the fall alarm when weight is removed from the pressure pad itself alerting the caregiver of the residents departure. Pads cannot be folded or immersed in any solution. The incontinent protected pressure pads can be wiped clean by using disinfectant wipes or anti-bacterial cleaner. To minimize spreading of infection, single patient use is recommended. For safety reasons pads must be replaced after recommended pad life has expired. The Fall Alarm System Monitor is the recommended monitor for use with these pads. This monitor features a Safety Auto-Reset™, "always on technology" which means there is no "off" switch. Other monitor features include a Nurse Call input plug, adjustable volume, three different alarm tones, and a programmable 0- 2 second delay. Fall Monitor ships with the Caregiver Key™ (Patent Pending), which can allow only the caregiver to silence the alarming fall monitor. This system can be configured for wireless annunciation on pocket pagers, wireless consoles or your existing master at your Nurse's Station (or other location) and displays the resident room that originated the alarm.

Call Today

Our experienced and knowledgeable technical staff would be happy to answer your questions or give you ideas on how to put wireless to work for your facility.



Phone: 1-888-826-3394
Fax: 1-208-762-4877
Website: www.wirelessnursecall.com

Address: Systemstechnologies
11310 N. Government Way
Hayden, ID 83835